

Shining® Scan Checklist for Scanning (Implants)

Make sure **Intraoral Mode** is enabled

Scan the pre-op **with** the tissue former/healing cap in place as a full arch

Scan the antagonist as a full arch

Scan the right-side bite

Scan the left-side bite

Add an anterior bite if necessary

*** Verify occlusal hits using the heat-map ***

Punch out the implant site

Remove the tissue former/healing cap and scan the gingival tissue

Punch out the implant site

Seat the scan body and take a PA to verify the proper seat

Scan the scan body